



ATMOS SCOPE HD Reprocessing Guide

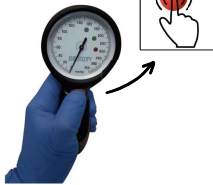


Please note that a PRE-CLEAN has already been performed by the clinical staff.

LEAK TESTING



IF A LEAK IS DETECTED, THE ENDOSCOPE MUST BE REPAIRED.

				
Ensure pressure compensation pin is applied to the pressure compensation valve	Squeeze bulb on pressure gauge until dial reaches 160	Deflect the scope up by sliding deflection lever down	Deflect the scope down by sliding deflection lever up	Press and hold red button until dial sits at '0'

MANUAL CLEANING



**DO NOT SQUEEZE, PULL, TWIST, OR TUG ON THE INSPECTION TUBE.
DO NOT SUBMERGE USB CONNECTOR.**

	
Submerge and wipe all external surfaces of endoscope and cleaning accessories for the time recommended by manufacturer	Ensure the lens is clean

RINSING

	In STERILE or DEIONIZED water, submerge and agitate water to rinse endoscope  2 MINUTES COMPLETE THE FINAL RINSE FOR THE FULL TWO-MINUTE DURATION. DO NOT SUBMERGE THE USB.
---	---

DRYING

		
Use soft, lint-free cloth or air compressor to remove excess moisture	Attach gas cap with card to pressure valve. Transport to reprocessing room in a covered container	Prepare the device for sterilization